

#900 –Single lens vials w/ screw cap

INDICATIONS: For storage of soft (hydrophilic)/rigid gas permeable/and hard contact lenses during chemical disinfection only. DO NOT USE WITH HEAT.

PREPARATIONS FOR USE: CLEAN WITH LENS CLEANING SOLUTION, RINSE, AND AIR DRY PRIOR TO USE.

DIRECTIONS FOR DAILY USE: CLEAN WITH LENS CLEANING SOLUTION, RINSE, AND AIR DRY DAILY. Place sufficient amount of solution in each vial prior to putting lenses in vials. Close lids carefully, make sure lids are fully seated on vials.

WARNING

Lens cases & vials can be a significant source of microbial contamination. To prevent eye infections, lens cases & vials should be cleaned, rinsed, and air dried every day; and replaced frequently. Use of this lens vial with heat may cause warpage. USE FOR CHEMICAL DISINFECTION ONLY. DO NOT USE WITH HEAT.

This package contains a chemical known to the State of California to cause birth defects or other reproductive harm.

Manufactured By:
Paragon Vision Sciences
2120 W. Guadalupe Rd., Ste. 112
Gilbert, AZ 85233